

NCI Flow Cytometry: Bone Marrow Aspirate


⌘ Before Collection Day:

- Request testing schedule:
 - Stetler-Stevenson, Maryalice stetler@mail.nih.gov
 - Yuan, Constance yuanc@mail.nih.gov
 - Wang, Hao-Wei hao-wei.wang@nih.gov

- CRIS order: **Flow Cytometry, Bone Marrow**
scheduling.com: **MORNING collection**

⌘ On Collection Day: **DELIVERY Cutoff 1:00 p.m.**

- Retrieve heparinized syringe (3SWN Procedures unit)
- In SoftID, print specimen label

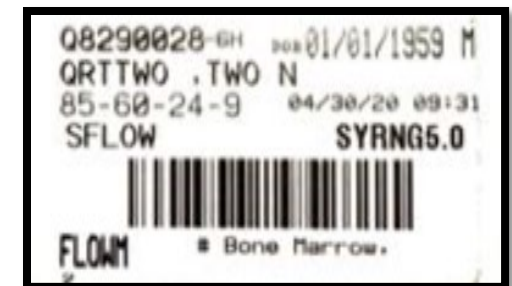

⌘ Transfer **3-5ml bone marrow aspirate** into heparinized syringe

- Draw small bubble into syringe (for mixing purposes)
- invert 5 times mixing heparin with aspirate material to prevent clotting

STAT Transport (dry) to Room 3S-240

Where can I find FLOW information?



⌘ Flow Cytometry specimen collection manual (online):

<http://home.ccr.cancer.gov/lop/Clinical/specimencollections.asp#flow>

⌘ Contact Supervisor: Catharine McCoy

Office : 301-480-8078

Flow lab: Room 3S-240